

LTL004: VICTOR READER STREAM TUTORIAL



Victor Reader Stream



What's covered in this tutorial

- Player Overview
- Player Requirements & Battery Information
- Connector Ports
- Power Control Key
- Adjustable Variable Control Keys
- Basic Navigation Features
- Advance Navigation Functions
- Special Keys
- Support Information



What's Included in the Box

- Victor Reader Stream
- User's Guide & Getting Started on CD
- Nylon Carrying Case
- AC Adapter
- Ear Buds
- 2 USB Cables (host & slave)
- Warranty Card




Player Requirements & Battery Information

- Requires User Authorization Key (UAK)
 - ▶ Preinstalled if purchased from RFB&D
- User replaceable rechargeable battery
- Battery automatically recharges when plugged into an electrical outlet
- Fully charged, provides 15 hours of book playtime
- 4 hours to charge battery

Note: additional battery available from manufacturer or an authorized dealer



Product Overview

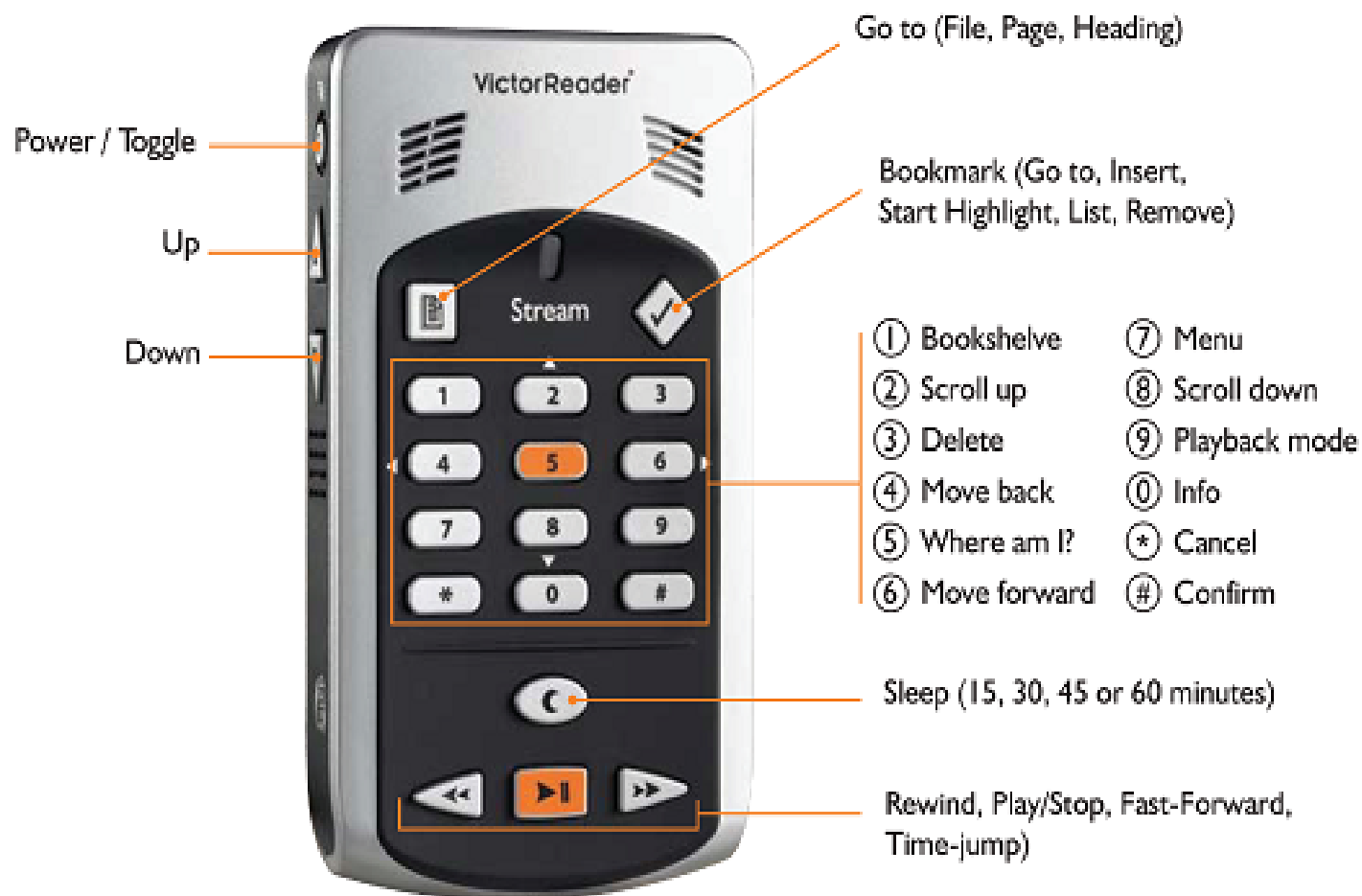

Connector Ports

Connector Ports

- Top Edge
 - ❖ AC adaptor port
 - ❖ SD memory card slot
 - ❖ Mini USB slot
- Right Side
 - ❖ Earphone port
 - ❖ Microphone port
 - ❖ Record button



Control Key: ON/OFF Key

Control Function Keys

ON/Off Power Key

- Located on the top left side
- Circle shape

To turn on, press and hold the power key, you will hear **ONE** beep

To turn off, press and hold the power key, you will hear **TWO** beeps



Adjustable Variable Keys: Volume - Speed & Tone Key

Control Function Keys

- Adjustable Controls

Slightly press the power button to toggle between

- Volume
- Speed
- Tone

Use the up arrow to increase the controls

Use the down arrow to decrease the controls



Basic Navigation Keys: Fast-forward - Rewind - Play/Stop

Play/Stop Key



- ▶ Press to start playing a book
- ▶ Press to stop playing a book

NOTE: Play/Stop can also be used to confirm an operation or end a numeric or text entry.

Rewind and Fast Forward Keys



- ▶ Use these keys to move forward or backwards quickly



Navigation Keys: 2, 4, 6, and 8

Navigation Keys: 2, 4, 6, and 8

Use these keys to move quickly from one level to another

Use keys 2 and 8 to choose a navigation level
(page, chapter, section, unit)

- ▶ **2 Key** = Moves forward to next level
- ▶ **8 Key** = Moves backwards to the next level

Use keys 4 and 6 to move backward and forward between levels or elements

- ▶ **4 Key** = moves forward between levels
- ▶ **6 Key** = moves backwards between levels



Numeric Keys

Numeric Key Functions

- ▶ **1: Bookshelf/Built-in user guide**
- ▶ **3: Delete**
 - ◆ Can delete entire book or subfolder containing book
- ▶ **5: Where Am I**
 - ◆ Announces reading position
 - heading numbers, section title
 - file name
- ▶ **0: Info**
 - ◆ Announces book, player, battery info
 - ◆ Also used to enter/exit Key Describer Mode
- ▶ ***:Cancel**
- ▶ **#:Confirm**



Advance Function: Go-To Page/Heading Key

Go-To Page: Square Shape Key



Go to Page

- ▶ Press the Go to Page key, then enter a page number to go directly to your desired page
- ▶ Press the # sign to confirm your operation

Go to Heading

- ▶ Press the Go-to-Page key twice, then enter a heading number relative to the start of your DAISY book
- ▶ Press the # sign to confirm your operation

Useful for books without page markers



Advance Function: Bookmarks

Bookmarks



- Quickly and easily return to a specific page
- Diamond Shape
- Unlimited number of bookmarks in one audiobook

You can

- Insert bookmarks
- Go to bookmark
- Hear a list of bookmarks
- Remove bookmarks



Advance Function: Bookmarks



Bookmarks

| | |
|-------------------------|---|
| Insert Bookmarks | Press Bookmark key until you hear “insert bookmark” Enter a number from 1 to 99998, press # |
| Go to Bookmark | Press Bookmark key until you hear “Go to bookmark” Enter the number of the bookmark, press # |
| List Bookmarks | Press Bookmark key until you hear “Bookmark list” Use key 4 or key 6 to go from one bookmark to the next Press # to confirm the current position & continue play |
| Remove Bookmarks | Press Bookmark key until you hear “Remove bookmark” Enter the number of the bookmark you wish to remove <i>Enter 99999# to remove all bookmarks</i> |

Advance Function: Bookmarks

Bookmarks



| | |
|--|---|
| <p>Start Highlight Bookmark</p> | <p>Press Bookmark key until you hear “Start highlight Bookmark”</p> <p>Enter a bookmark number</p> <p>Press # sign or PLAY/STOP key to confirm</p> |
| <p>End Highlight Bookmark</p> | <p>After setting the starting position, navigate to the point where you wish to mark the end of the highlight.</p> <p>Press the Bookmark key until you hear “End Highlight Bookmark”</p> <p>Press the # sign or PLAY/STOP key to confirm</p> <p>Note: If the end position is placed before the start position, they will be switched. The message “Highlight Bookmark “X” Inserted” is then played.</p> |

Advance Function: Sleep Mode

Sleep Mode

Program a delay after which the player automatically shuts-off

Press the Sleep Mode key to shutdown delays of:

- ▶ 15
- ▶ 30
- ▶ 45
- ▶ 60

Sleep Key





RFB&D

Victor Reader Stream Tutorial

www.rfbd.org

Member Service 800-221-4792

custserv@rfbd.org